Ref.: ZZM001EU
ZZM002EU
ZZM003EU
EN149:2001
C0070

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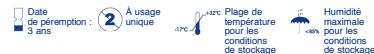
② Description and indications :

MacoPharma Respiratory Protection Masks FFP1, FFP2 and FFP3 for single use meet the requirements of standard EN149:2001 and French recommendation dated 28 September 2005, whose purpose is to check filtration performance after a test of exposure to 120 mg spray. These filtering masks are recommended to any person (health care professionals, patients, visitors...) looking for a protection against liquid and solid sprays.

Instructions for use :

This mask may only be used by people who were trained to use it. Take the mask out of its pack at the time of using it only.

Before using it, ensure that it is intact and that elasticised strings are intact too.

To ensure that the respiratory mask tightly adheres to your face :

- Cup your hands over the mask to block up its surface of filtration as much as possible, without moving it.
 - Breathe out really strongly :
 - if any air flows around your nose, readjust the nose bar as in picture 7
 - if any air flows out around the mask, carefully adjust the strings to the back.
- In case tightness obtained is not satisfactory, do not enter the area where it is required to wear the mask.

Warning :

- It is unlikely that tightness requirements are met if hairs from your beard stick out under the face seal.
- Leave the premises if :
 - you feel dizzy or feel any pain ;
 - it becomes hard to breathe.

Dispose of and replace the mask if :

- it is damaged ;
 - it seems to hamper your breathing ;
 - it is tainted by blood or any other infectious substance.
- Dispose of the mask via the Infectious Hazard Care Activity Waste circuit.

Use restrictions :

This mask does not provide any protection against gases or fumes. Do not use this mask if the atmospheric pollutant concentration exceeds 9 times its VLE (exposure limit value). This half mask does not supply any oxygen. Do not use in atmospheres which do not contain at least 17% oxygen. Do not use in an explosive atmosphere.

This mask does not require any maintenance. It has been designed for one single continuous work session, i.e. for 8 hours as a maximum, in compliance with instructions as worded in the Journal Officiel de la République Française, 28 September 2005.

Any failure to comply with instructions and warnings concerning the use of this respiratory protection mask for the whole period of exposure may reduce its efficiency. This product does not eliminate the risk of contracting diseases or infections.

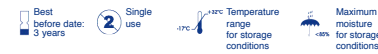
Conformities :

This mask meets requirements defined by European Directive 89/686/EEC. It has been examined by the Institut National de Recherche et de Sécurité, Avenue de Bourgogne 54 500 Vandœuvre, France (identification number: 0070). This mask meets extra requirements as stipulated by the Journal Officiel de la République Française, 28 September 2005.

The respiratory protection mask FFP1, FFP2 and FFP3 do not contain any latex.

Storage :

Store according to recommendations as displayed on the pack.

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